

# th Dermacol

---

around the base of the stoma, preventing skin from becoming red, itchy and sore. If you're the parent of a child with a stoma or you're a teenager with a stoma yourself, this leaflet will show you how Dermacol could work for you.

---





### Soft and flexible collar

Thin and flexible, Dermacol bends, twists and turns with active bodies, staying flush to the skin without forming creases.

### Easy to apply

There are two easy ways to attach Dermacol. Use this leaflet to find the best method for you.




Validating Salts research into healthy stoma skin

### Skin-friendly wafer

Accredited by the British Skin Foundation, the hydrocolloid wafer is perfect for young, delicate and sensitive skin.

### Ideal size and height

Dermacol is the perfect height and width to fit stomas of all shapes and sizes – even babies! *Don't worry if the collar is slightly too long – it can simply be trimmed. First, snip into the collar at intervals, then trim around the collar to get a perfect length.*

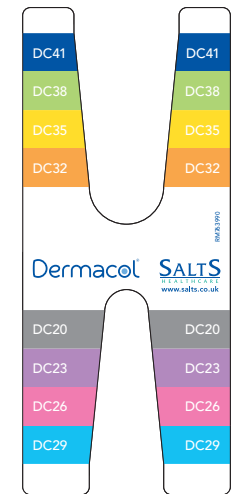


### 8 colour-coded sizes

Use the simple colour coded sizing guide to help you find the correct Dermacol product for a perfect fit.

### Strong yet gentle

The adhesive is strong but very soft. It absorbs mucus from the stoma and sweat from the skin, staying comfortable at all times.

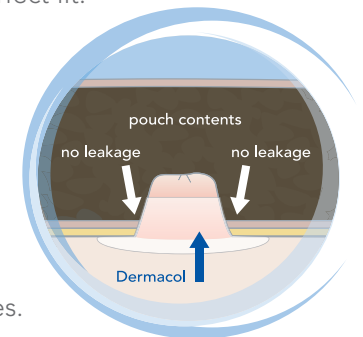


# Dermacol for babies

The area of skin around your baby's stoma is likely to be sensitive and, if not kept clean and dry, can become sore. Dermacol is made from soft material that will wriggle with your baby and has a skin-friendly hydrocolloid to protect even the most delicate skin.


*"She's stopped getting rashes, thanks to Dermacol."*




## How to fit Dermacol on your baby

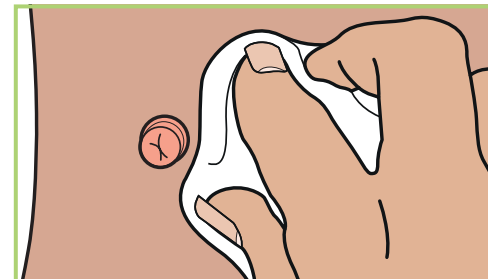


1 Ensure that the skin around the stoma is clean and dry.

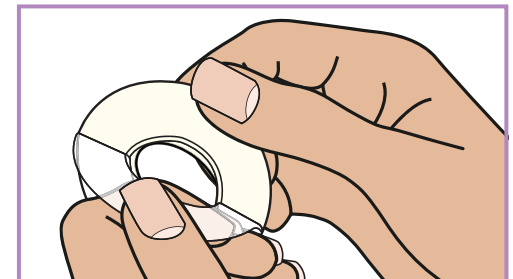

2 Remove film backing.

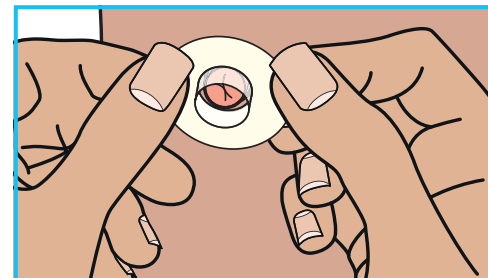


3 Slide collar gently up and around the stoma.

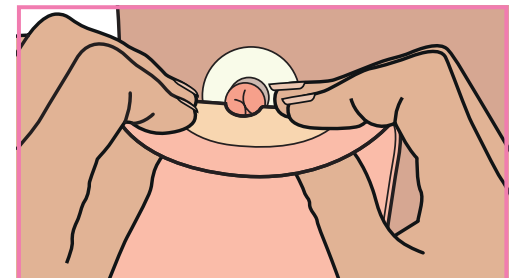

4 Securely connect the pouch or flange to the collar. If collar pushes down the stoma, try method 2 or try cutting the flange slightly larger. Put the pouch on in the usual way.



5 Smooth round with fingers to make sure the pouch is secure. If using a two piece system apply pouch to flange.

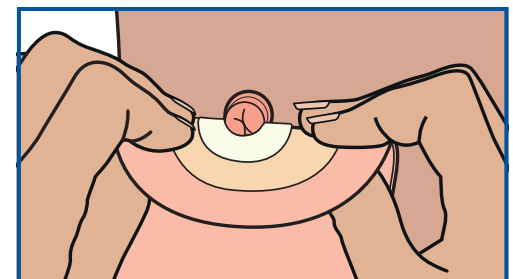


6 Gently remove pouch or flange together with the collar.


# Dermacol for small children

With Dermacol, leaks are one less thing to worry about. The soft collar is designed to bend and flex with even the most active children, providing a leakproof barrier that will keep skin smooth and soft all day long.


*"He's a lot happier since we started using Dermacol."*

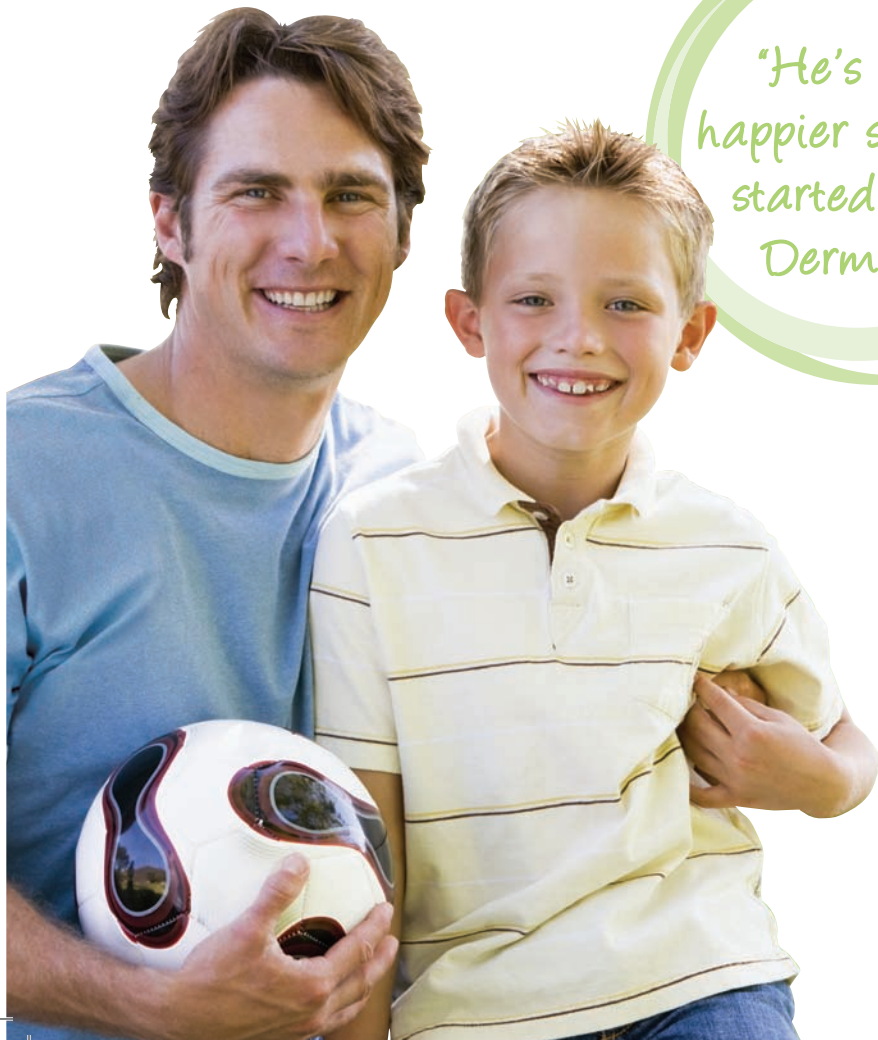


## How to fit Dermacol on your child

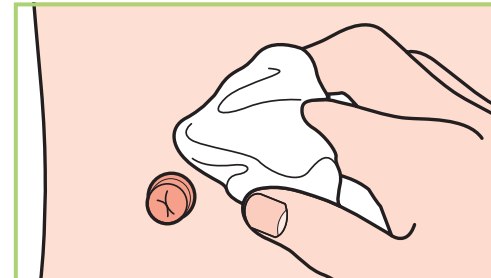


1 Ensure that the skin around the stoma is clean and dry.

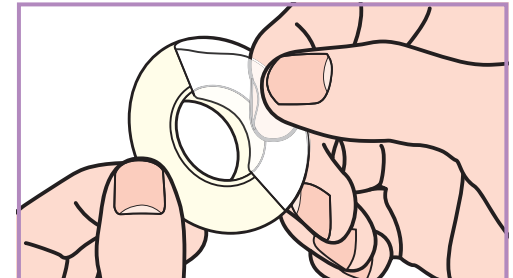


2 Remove film backing.

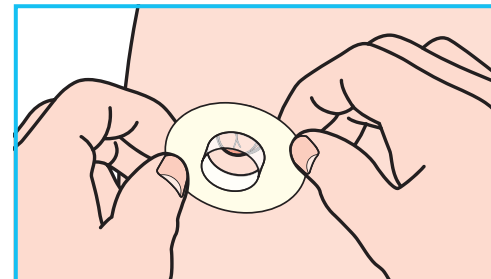


3 Slide collar gently up and around the stoma.

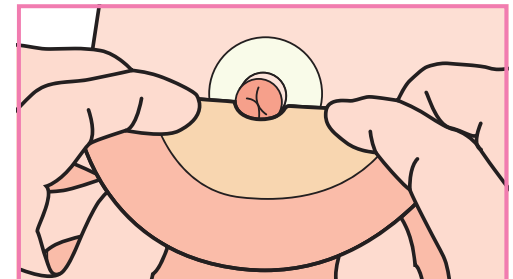

4 Securely connect the pouch or flange to the collar. If collar pushes down the stoma, try method 2 or try cutting the flange slightly larger. Put the pouch on in the usual way.

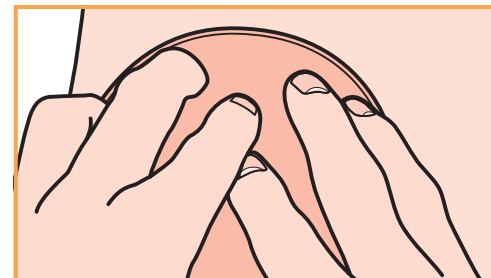

5 Smooth round with fingers to make sure the pouch is secure. If using a two piece system apply pouch to flange.

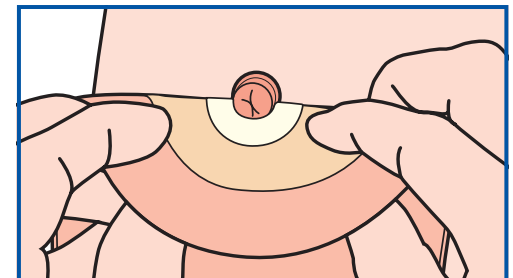

6 Gently remove pouch or flange together with the collar.

# Dermacol for teenagers

As a teenager, your body shape is constantly changing, which can sometimes lead to ill-fitting appliances and leaks. Dermacol has been specifically designed to adapt to these changes and help you to stay leak-free in whatever you do.

*"Dermacol fits all shapes, sizes and types of stomas so can help you stay leak-free in whatever you do."*

