

# SecuPlast<sup>®</sup>

## Mouldable Seals

New SecuPlast Mouldable Seals help prevent sore skin by reducing the leaks that are the main cause of it.

Simply mould SecuPlast Mouldable Seals to the exact size and shape of your stoma and the exact contours of your abdomen to create a customised, comfortable and leak proof seal.

- Suitable for any make or style of pouch and all stoma types
- Very easy to mould and shape to fit; then place in position
- Very easy to rejoin if it breaks
- Easy to break into smaller pieces, if required
- Skin friendly, with no sting, even on sore skin
- Customised comfort – SecuPlast Mouldable Seals keep the shape you create for secure and lasting protection from leaks.



### Thin

Diameter: 48mm Thickness: 3mm  
Quantity per pack: 30




### Standard

Diameter: 48mm Thickness: 4.2mm  
Quantity per pack: 30





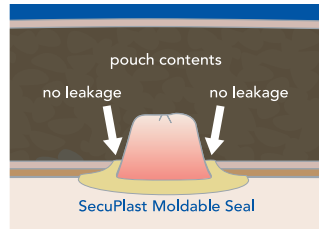

### Large

Diameter: 98mm Thickness: 3mm  
Quantity per pack: 10



SecuPlast Mouldable Seals stop leakage by forming a secure, low profile seal around your stoma



SECUPLAST <sup>®</sup> HYDRO and SECUPLAST <sup>®</sup> MOULDABLE SEALS	
	<b>Hydro</b> SIZE: One size fits all QUANTITY: 30 ORDER CODE: SPH1
	<b>Thin</b> DIAMETER: 48mm THICKNESS: 3mm QUANTITY: 30 ORDER CODE: SMST
	<b>Standard</b> DIAMETER: 48mm THICKNESS: 4.2mm QUANTITY: 30 ORDER CODE: SMSS
	<b>Large</b> DIAMETER: 98mm THICKNESS: 3mm QUANTITY: 10 ORDER CODE: SMSL

For more information and free samples CALL TOLL FREE 1300 784 737 or visit [www.ainscorp.com.au](http://www.ainscorp.com.au)



Validating Salts research into healthy stoma skin



[www.salts.co.uk](http://www.salts.co.uk)



Validating Salts research into healthy stoma skin



Excellence in stoma care

SecuPlast<sup>®</sup>  
Skin-friendly hydrocolloid strips and mouldable seals

STOMA SOLUTIONS



EXTRA SECURITY

Adhesion, leakage and skin problems affect more people with stomas than you might think. Yet they're not an inevitable part of living with a pouch.

**SecuPlast Hydro** skin-friendly, hydrocolloid security strips and **SecuPlast Mouldable Seals** provide a reliable, effective way to secure your pouch and prevent leaks so you can feel comfortable and confident.

## SecuPlast<sup>®</sup> Hydro

Security has never been more simple. Or more effective. SecuPlast Hydro strips are the thin, skin-friendly, hydrocolloid alternative to sticky and cumbersome tape.

In fact, over 95% of people who tried SecuPlast Hydro strips described it as 'comfortable'

- Extra security with any make and type of pouch
- One size fits all
- Easy to use – precision positioning and less wastage
- Non-allergenic – suitable for patients who can't use microporous tapes
- Wider than other similar products, providing even more security around the flange.



One size fits all  
Quantity per pack: 30



## How to use SecuPlast Hydro strips

Just follow these simple steps.  
It couldn't be more straightforward:



1 Remove release film.



2 Position as required and smooth strip onto skin. Use as many pieces as you need.

*96% of patients said SecuPlast Hydro was comfortable and*

*62% found SecuPlast Hydro better than their usual product*

## How to use SecuPlast Mouldable Seals

SecuPlast Mouldable Seals are so easy to apply. They will not lose their stickiness so you can handle them as much as you like:

*"This product has proved excellent in all ways, giving me confidence in security against leakage."*

*"I was experiencing leaks mainly at night when I was not wearing a corset. Your product seems to have virtually eliminated the problem. Thank you."*

*"These seals have prevented leaks and help keep my pouch on for longer."*




1 Remove release film from both sides.



2 Mould into shape to suit your personal requirements.



3 Can easily be rejoined if required...

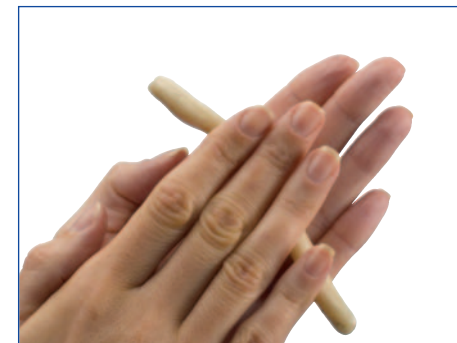

4 ...rolled between the palms of the hand...



5 ...to make a thicker seal...



6 ...or roll from the centre to create a thicker lip.