



WITH EXTRACTS OF  
ALOE VERA



Recognising Salts research  
into healthy stoma skin



DERMATOLOGICALLY  
ACCREDITED



# Changing lives with superior skin protection

INFORMATION AND PRODUCT SELECTION GUIDE  
FOR HEALTHCARE PROFESSIONALS



# SALTS MOULDABLE SEALS

HIGH ABSORPTION AND ADHESION



## Features:


- ▶ Help to create a customised, comfortable, leak-resistant seal
- ▶ Available with and without extracts of Aloe Vera
- ▶ Aloe Vera extracts can help soothe and protect (SMSA only)
- ▶ Mould to the exact shape and size of the stoma
- ▶ Absorb and lock moisture away from the skin for a longer period of time, helping to prevent output from causing irritation
- ▶ Adhere to wet skin, thanks to their superior stickiness
- ▶ Stay in place, even when the body twists and turns
- ▶ Can be stretched, broken and reshaped without the need for any warming
- ▶ Many sizes available: SMSA – 5 sizes; SMS – 3 sizes
- ▶ NEW 35–70mm size to cater for larger or loop stomas (SMSA only)
- ▶ Suitable for all stoma types




## HOW TO APPLY




1 Clean and dry the area around the stoma to make sure it's free from grease or creams.

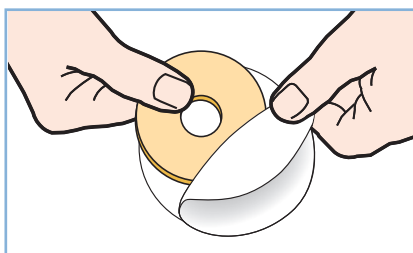


2 Carefully remove the paper from both surfaces of the mouldable seal.

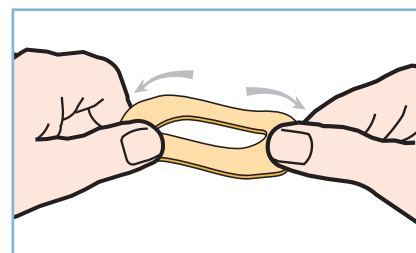


3 Use your fingers to gently mould the seal to the required shape. Either side can be placed against the skin.

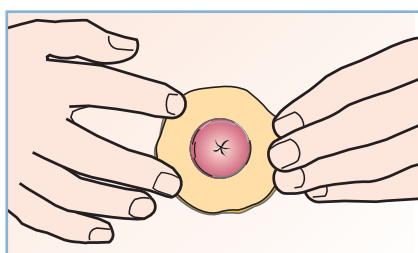

4 Carefully mould the seal so it fits snugly, with all skin directly around the stoma covered.

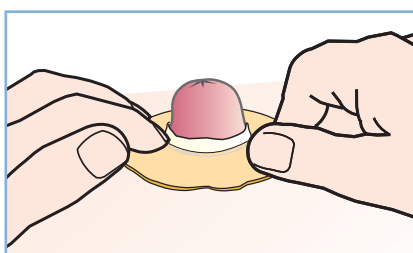


5 Flatten the outer edge of the seal with your fingertips.

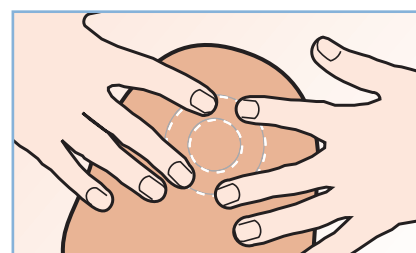

6 **One-piece users** Put the pouch over the seal and briefly hold it in position.  
**Two-piece users** Put the flange over the seal, then attach the pouch as usual.

## MOULD INTO SHAPE TO SUIT YOUR PERSONAL REQUIREMENTS

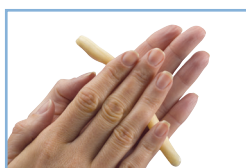

### SALTS MOULDBLE SEALS WITH ALOE

SIZE	DIAMETER	THICKNESS	ORDER CODES	PACK SIZE
Thin	15mm – 40mm	3mm	SMSAT15	30
Thin	35mm – 70mm	3mm	SMSAT35	30
Standard	15mm – 40mm	4.2mm	SMSAS15	30
Standard	35mm – 70mm	4.2mm	SMSAS35	30
Large	15mm – 100mm	3mm	SMSAL	10

### SALTS MOULDBLE SEALS

SIZE	DIAMETER	THICKNESS	ORDER CODES	PACK SIZE
Thin	50mm	3mm	SMST	30
Standard	50mm	4.2mm	SMSS	30
Large	100mm	3mm	SMSL	10


# SALTS ALOE RINGS




## MODERATE ABSORPTION AND ADHESION

### Features:


- ▶ Skin-friendly, thin and flexible hydrocolloid rings
- ▶ Aloe Vera extracts to help soothe and protect
- ▶ Quick and easy to apply
- ▶ No need for any initial warming or moulding – simply apply straight over the stoma
- ▶ Discreet and low-profile, at only 1.8mm thick
- ▶ Can be stretched to accommodate moderately oval stomas
- ▶ Available in 4 sizes
- ▶ Suitable for all stoma types

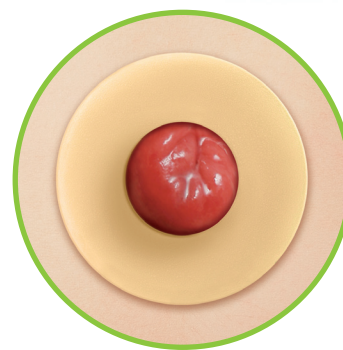


### HOW TO APPLY

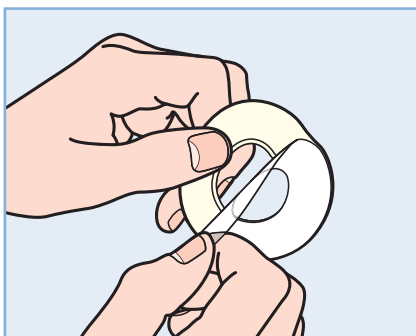

1 Remove the film backing from the ring. No initial warming or moulding is required.

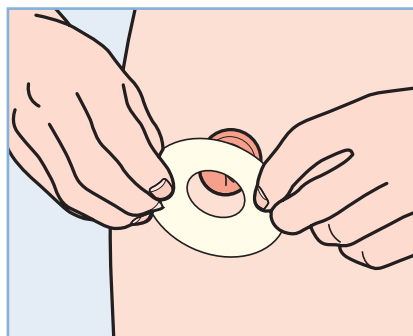

2 Place the ring around the stoma, then smooth round it with the fingers.

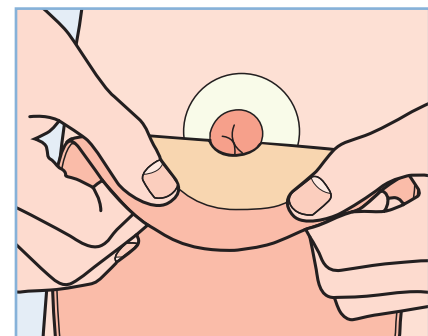

3 Place the pouch gently over the stoma and ring, and smooth all round it with the fingers.


### SALTS ALOE RINGS

INTERNAL DIAMETER	ORDER CODES	PACK SIZE
25mm	SAR25	30
32mm	SAR32	30
38mm	SAR38	30
45mm	SAR45	30

# SALTS STOMA COLLAR



## SUPERIOR SKIN PROTECTION

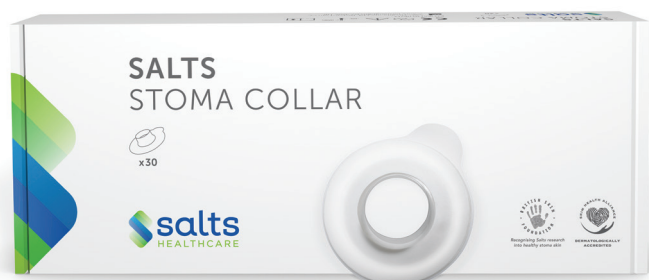


### Features:

- ▶ Provides a leak-proof barrier around the base of the stoma
- ▶ Helps to prevent output from coming into contact with the skin
- ▶ Extremely soft collar material twists and turns with the movement of the body
- ▶ Collar can be trimmed to size
- ▶ Fits moderately oval stomas
- ▶ Suitable for all stoma types
- ▶ Soft, flexible collar with a skin-friendly hydrocolloid
- ▶ Works with flush stomas, when used in conjunction with convex bags



### HOW TO MEASURE

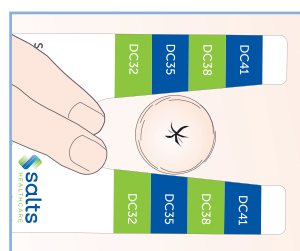


- Carefully slide the guide up and around the stoma. Stop when the edges of the size guide touch the stoma.
- Read the collar number from the chart. This is the ideal collar size. Check the stoma size regularly.


### HOW TO APPLY




- Clean and dry the area around the stoma to make sure it's free from grease or creams.

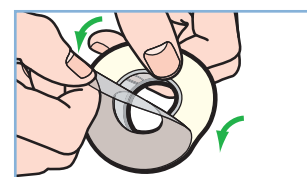


- Remove film backing from the collar – this is indicated by the side the code is printed on.

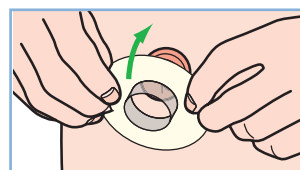

- Place collar around stoma, then smooth all round with the fingers.

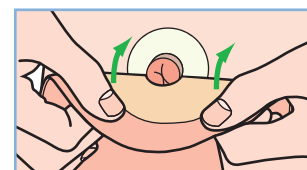

- One-piece users:** Place pouch gently over the stoma and collar, ensuring the collar does not get caught.

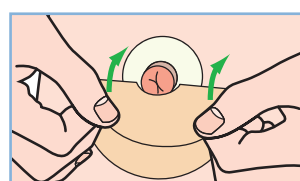

- Two-piece users:** Place flange gently over the stoma and collar, ensuring the collar does not get caught.



- Smooth all round with the fingers.

### SALTS STOMA COLLAR

STOMA SIZE	ORDER CODES	PACK SIZE	STOMA SIZE	ORDER CODES	PACK SIZE
17mm – 20mm	DC20	30	30mm – 32mm	DC32	30
21mm – 23mm	DC23	30	33mm – 35mm	DC35	30
24mm – 26mm	DC26	30	36mm – 38mm	DC38	30
27mm – 29mm	DC29	30	39mm – 41mm	DC41	30

# SALTS FLANGE EXTENDER WITH ALOE



## PICTURE FRAMING FOR EXTRA SECURITY

### Features:

- ▶ Helps secure the edges of the wafer, providing added security when needed.
- ▶ Aloe Vera extracts to help soothe and protect
- ▶ One size fits all
- ▶ 120° – use as many strips as required (3 strips will form a full circle)
- ▶ Extra security with any type of bag
- ▶ Non-allergenic – great for people who can't use microporous tapes



### HOW TO APPLY



1 Remove the film backing.



2 Position as required and smooth onto the skin.

SALTS FLANGE EXTENDER WITH ALOE		
SIZE	ORDER CODE	PACK SIZE
One size fits all	SPHA2	30

# SALTS HYDROTAPE Hydrocolloid


## PICTURE FRAMING FOR EXTRA SECURITY

**Features:**


- ▶ Thin and flexible hydrocolloid
- ▶ Extra security with any make and type of bag
- ▶ 'Picture framing' around appliance
- ▶ Non-allergenic – suitable for patients who can't use microporous tapes
- ▶ Simply cut to required length
- ▶ Easy to use – precision positioning and less wastage




### HOW TO APPLY

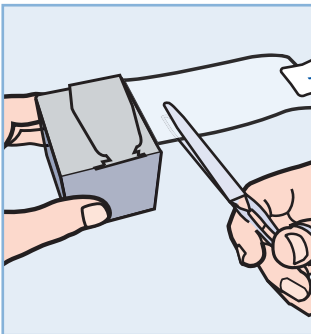


1 Cut the hydrocolloid tape to the required length.

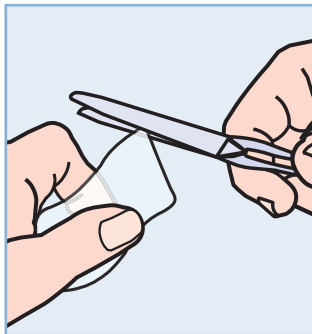

2 Slightly round the ends to prevent the hydrocolloid from rolling back.

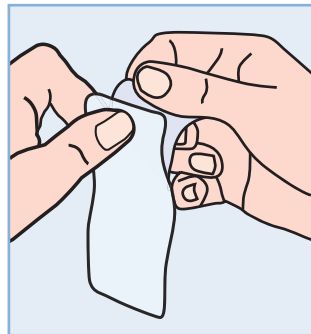

3 Remove the film backing.

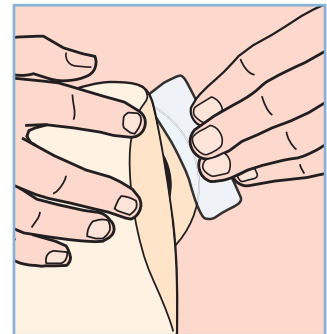

4 Position as required and smooth onto the skin.

### SALTS HYDROTAPE Hydrocolloid

**SIZE**

1.5m long x 25mm wide

**ORDER CODE**

HT2

**QUANTITY**

1

# SALTS NO-STING STOMA PASTE




## HIGHLY ABSORBENT HIGH TACK FILLER

### Features:


- ▶ Provides an ideal surface for the bag to adhere to
- ▶ Skin-friendly and alcohol-free
- ▶ Fills gaps and crevices
- ▶ Helps to create a more secure seal on uneven skin
- ▶ Reduces leakages and protects skin, resulting in an extended wear-time

### HOW TO APPLY

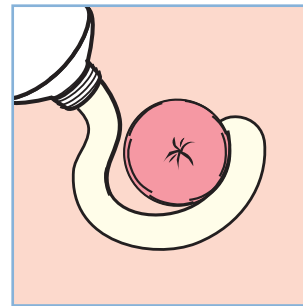

#### One-piece users:

Apply the paste around the stoma where any gaps or crevices are known to cause leaks.

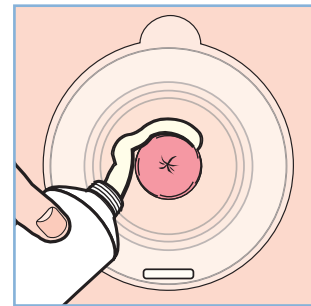

#### Two-piece users:

Apply the wafer as usual, then apply the paste to any gaps between the stoma and the edge of the hole in the wafer.



### SALTS NO-STING STOMA PASTE

SIZE	ORDER CODE	QUANTITY
60g	NSP1	1

### For free samples, please contact:



**TOLL FREE 1300 784 737** (Australia)

**TOLL FREE 0800 100 146** (New Zealand)



**ostomy@ainscorp.com.au**



**www.ainscorp.com.au**



@SaltsHealthcare



SaltsHealthcare



® Registered trade marks of Salts Healthcare Ltd. © Salts Healthcare Ltd 2018. Products and trade marks of Salts Healthcare Ltd are protected by UK and foreign patents, registered designs and trade marks. For further details, please visit [www.salts.co.uk](http://www.salts.co.uk)  
RM765334 04/18