

SALTS HYDROTAPE Hydrocolloid


PICTURE FRAMING FOR EXTRA SECURITY

Features:


- ▶ Thin and flexible hydrocolloid
- ▶ Extra security with any make and type of bag
- ▶ 'Picture framing' around appliance
- ▶ Non-allergenic – suitable for patients who can't use microporous tapes
- ▶ Simply cut to required length
- ▶ Easy to use – precision positioning and less wastage




HOW TO APPLY

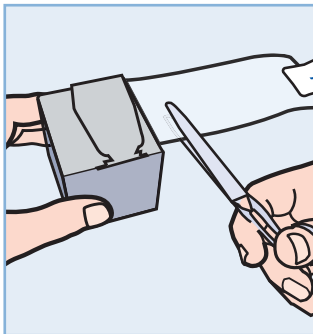

1 Cut the hydrocolloid tape to the required length.

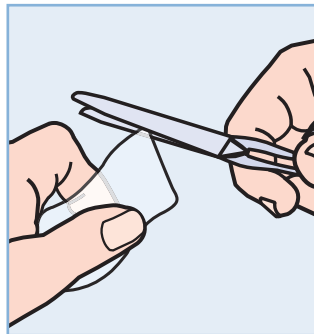

2 Slightly round the ends to prevent the hydrocolloid from rolling back.

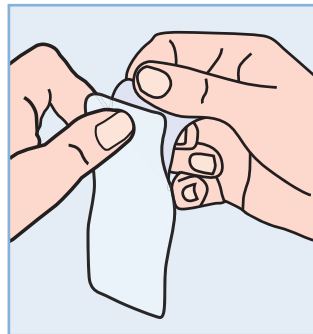

3 Remove the film backing.



4 Position as required and smooth onto the skin.

SALTS HYDROTAPE Hydrocolloid

SIZE

1.5m long x 25mm wide

ORDER CODE

HT2

QUANTITY

1