


# SALTS STOMA COLLAR



## SUPERIOR SKIN PROTECTION



### HOW TO MEASURE

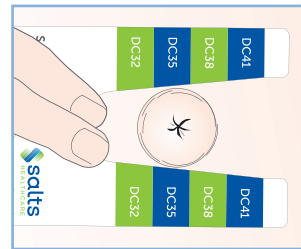

- A** Carefully slide the guide up and around the stoma. Stop when the edges of the size guide touch the stoma.
- B** Read the collar number from the chart. This is the ideal collar size. Check the stoma size regularly.

### Features:


- ▶ Provides a leak-proof barrier around the base of the stoma
- ▶ Helps to prevent output from coming into contact with the skin
- ▶ Extremely soft collar material twists and turns with the movement of the body
- ▶ Collar can be trimmed to size
- ▶ Fits moderately oval stomas
- ▶ Suitable for all stoma types
- ▶ Soft, flexible collar with a skin-friendly hydrocolloid
- ▶ Works with flush stomas, when used in conjunction with convex bags




### HOW TO APPLY




- 1** Clean and dry the area around the stoma to make sure it's free from grease or creams.

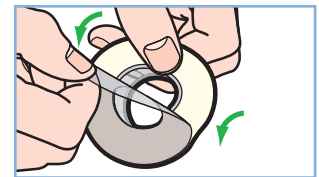


- 2** Remove film backing from the collar – this is indicated by the side the code is printed on.

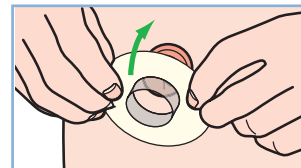

- 3** Place collar around stoma, then smooth all round with the fingers.

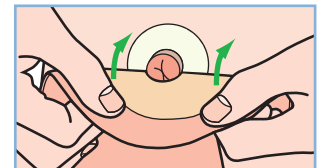

- 4 One-piece users:** Place pouch gently over the stoma and collar, ensuring the collar does not get caught.

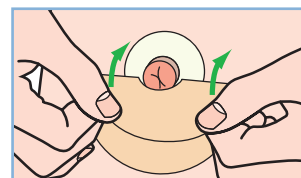

- 4 Two-piece users:** Place flange gently over the stoma and collar, ensuring the collar does not get caught.



- 5** Smooth all round with the fingers.

### SALTS STOMA COLLAR

STOMA SIZE	ORDER CODES	PACK SIZE	STOMA SIZE	ORDER CODES	PACK SIZE
17mm – 20mm	DC20	30	30mm – 32mm	DC32	30
21mm – 23mm	DC23	30	33mm – 35mm	DC35	30
24mm – 26mm	DC26	30	36mm – 38mm	DC38	30
27mm – 29mm	DC29	30	39mm – 41mm	DC41	30